

Central Sterile & Supply Department
PT. B.D.SHARMA PGIMS ROHTAK

SOP No.: CSSD/SOP/08

Issue Date:

Review Before:

***Title: SOP FOR PACKING AND STERILIZATION OF
DRUMS OF SURGERY WARDS.***

OBJECTIVES:

To lay down a procedure for packing and sterilization of surgical drums.

SCOPE:

This procedure is applicable to packing and sterilization of drums from various surgery wards.

RESPONSIBILITY:

Assistant Supervisor, Sterilisation Technician, CSSD Attendants.

PROCEDURE:

1. Receive the empty drums from various surgery wards and maintain the record in register.
2. Open the drums and fill them with gauze piece and cotton (cut by gauze cutting machine) as per required size and quantity.
3. Close the lid of drums and slide the shutter to perforated side before sterilization .
4. After sterilization drums perforated site shifts to close shutter site and drums are stored in sterile store.
5. Issue the drums through issuing window after taking the signature of respective department.

	Prepared By	Checked & Approved by	Authorized By
Designation	St. technician	Asstt. Supervisor	Officer In charge
Name	Anil Kumar	Yogesh Kumar	P.K. Verma
Sign/ Date			